



Client's Name:
PHIN:
Resource Coordinator:
Office Location:

HOME CARE ATTENDANT ASSIGNMENT TASK TRAINING RECORD - Intermittent Pneumatic Compression			
HCA Name: _____			
Date Trained: _____			
Steps to review with Home Care Attendant	Met	Not Met	Comments
1. Wash hands.			
2. Apply disposable gloves.			
3. Remove pump from packaging with garment(s) for use.			
4. Place pump into wall outlet. DO NOT switch on.			
5. Place pump flat on floor or by hanging device.			
6. Push long tube from garment onto air port at side of pump. On some models, if only one garment is used the provided stopper must block the unused port.			
7. Make sure that power is off, remove garment from its wrapping and unzip it.			
8. Inspect skin for abrasions or rash.			
9. Place arm or leg into garment and zip it up. Ensure garment is fully zipped closed. Garment will conform to contours of a limb.			
10. Ensure that pressure dial on pump is set at a minimum level. Turn power switch on and increase pressure to required level.			
11. Remove disposable gloves and discard.			
12. Wash hands.			
13. Document correctly on the Home Care Medication Assignment Record – Home Care Attendant.			
Cleaning Instructions:			
1. The pump can be cleaned using a soft cloth and water. DO NOT wash pump while it is plugged into the wall outlet.			
2. Garment can be washed in luke warm water using normal detergent or soap powder. Before washing garments, be sure to seal off both ends of the garment air input using your connector tubing. Place one end of tubing on one hole and connect other end to the other hole, forming a complete loop. This will prevent water from getting in garment during washing.			
3. Do not iron or dry clean the garment.			

Nurse Signature: _____

Home Care Attendant Signature: _____

FORWARD TO NURSE EDUCATOR – CLIENT SPECIFIC SERVICES WITHIN 24-48 HOURS

Procura Entry Completed

Reviewed by: _____