


06 May 2024

## **\*\*B. Braun Infusomat® Space® Large Volume Pump Sets\*\***

This information is to provide awareness of quality improvements made by B. Braun to their Infusomat® Space® large volume pump sets:

- Product packaging identifies NEW priming volume (images unavailable)
- Priming volumes have increased by 3ml (no immediate software updates will occur at this time)
- Tubing color change from slight blue hue to slight yellow hue
- Roller clamp tubing holder is less pronounced

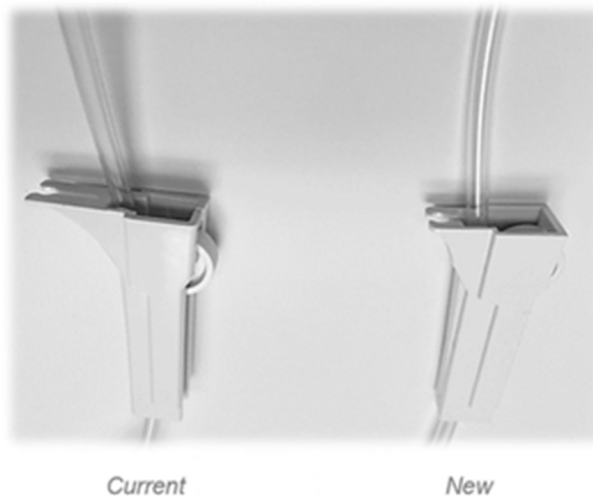

Please see the following page for the March 2024 B. Braun letter that lists the product codes for the impacted pump sets and the estimated timeline for distribution.

March 2024

**Priming Volume Changes for the B. Braun Infusomat® Space® Large Volume Pump Sets**

Dear Valued B. Braun Customer,

We are writing to inform you there is an upcoming change to the Infusomat® Space® large volume pump sets. This change will not affect the reference numbers of the current sets and therefore will not affect purchase orders.

The tubing inner and outer diameter will increase in size and as a result the priming volume will increase as specified in the table below. In addition, some slight visual changes may be noticeable in the new design including a new roller clamp and a slight color change of the tubing.

Please note, this revision in priming volume may not align with the current priming volume settings in your configuration file. If priming is activated, you may need to utilize the repeat priming option on the pump after the initial priming or after an air-in-line alarm. If you are interested in reviewing or updating your configuration file settings, please contact your B. Braun Sales Representative and refer to the complete instructions for use.

The following timeline is the current estimate to begin receiving product after the planned change in production. Please note this is an estimate and product may be received before or after this timeline with distribution:

Estimate to Receive Product:	Reference Number:	Product Description:	Current Priming Volume:	Revised Priming Volume:
May	490100	UNIV. 15 DROP PUMP SET W 3 CARESITE L.L.	23 mL	26 mL
May	363430	UNIV. 15 DROP PUMP SET, 3 CARESITE, ASV	23 mL	26 mL
June	490102	UNIV. 15 DROP PUMP SET W/2 CARESITE CV	22 mL	25 mL
June	363421	UNIV. 15 DROP PUMP SET, 2 CARESITE, ASV	22 mL	25 mL
July	363423	UNIV. 60 DROP PUMP SET, 2 CARESITES, ASV	21 mL	24 mL
July	490104	UNIV 60 DROP PUMP SET, W/2 CARESITE INJ.	21 mL	24 mL
July	363422	60 DROP METRISET PUMP, 3 CARESITES, ASV	21 mL	24 mL
July	490101	60 DROP METRISET PUMP SET, 3 CARESITES	21 mL	24 mL
July	362911	ADD ON PUMP SET W/ CARESITE, LUER LOCK	14 mL	16 mL
July	362031	UNIV. 15 DROP PUMP SET W/ CARESITE	22 mL	25 mL
July	362420	SPACE PUMP SET 15 DROP W/2 SAFEDAY	22 mL	25 mL
July	363433	UNIV. 15 DROP PUMP SET, 3 CARESITE, ASV	22 mL	25 mL
July	480254	SPACE PUMP IV SET 15D, 110 IN.	22 mL	25 mL
July	480263	SPACE PUMP IV SET 15D, 90 IN.	18 mL	20 mL
July	362035	INFUSOMAT SPACE PUMP IV SET	22 mL	25 mL
July	363904	UNIV. 15 DROP PUMP SET, ASV	22 mL	25 mL
July	480264	SPACE PUMP IV SET, 15 DRPS/ML, 95 IN.	18 mL	21 mL
July	363230	UNIV. 15 DROP PUMP SET, 3 ULTRASITE, ASV	22 mL	25 mL

B. Braun is dedicated to providing you with high quality products, service, and support. If you have any additional questions, please contact your local B. Braun Representative or B. Braun Customer Service at **1-800-227-2862** or email [Customer\\_Service@bbraunusa.com](mailto:Customer_Service@bbraunusa.com).

Thank you,



Jessica Andrews  
Group Product Director, Space® Infusion System